

<p><b>3M</b> Contact your tech representative at 3M to add "Perio Protect Lab" to your menu of available labs.</p>	<p><b>Sirona Cerec</b> Add Perio Protect, LLC as a Favorite Lab in your system:</p> <ul style="list-style-type: none"> <li>• In the SIRONA Connect heading, go to <a href="#">MY FAVORITE LABORATORIES</a>.</li> <li>• Click the <a href="#">SEARCH</a> option.</li> <li>• Enter our zip code: <a href="#">63125</a>.</li> <li>• Select <a href="#">Perio Protect, LLC</a>.</li> </ul>
<p><b>3Shape / Trios</b> Add Perio Protect Lab as a preferred lab:</p> <ul style="list-style-type: none"> <li>• Log in to the 3Shape website.</li> <li>• Click <a href="#">CONNECTIONS</a> in the navigation bar at the top.</li> <li>• In the <a href="#">CONNECT to PRACTICE</a> or <a href="#">LAB</a> window, type our email address: <a href="mailto:3D@Perio-Protect.com">3D@Perio-Protect.com</a></li> </ul> <p>Once we accept, you may start sending cases. Always include the Rx and periodontal charting in <a href="#">CASE DETAILS</a>.</p>	<p><b>Sirona PrimeScan</b> Add Perio Protect, LLC as a Favorite Contractor:</p> <ul style="list-style-type: none"> <li>• Go to: <a href="http://Customer.ConnectCaseCenter.com">Customer.ConnectCaseCenter.com</a> and log in.</li> <li>• Under <a href="#">PARTNER</a> (on the left), choose <a href="#">MY FAVORITE CONTRACTOR</a>.</li> <li>• Search for us by Perio Protect or by our zip code: <a href="#">63125</a>.</li> <li>• Add <a href="#">Perio Protect</a>.</li> </ul> <p>For more technical support, contact the company where you purchased your scanner or call Dentsply Sirona's Helpdesk at 1-800-659-5977.</p>
<p><b>CareStream / CS Connect</b> Invite Perio Protect Lab to be a laboratory partner:</p> <ul style="list-style-type: none"> <li>• Log in to CS Connect as a dentist.</li> <li>• In the tool bar, click this icon: </li> <li>• In the <a href="#">PARTNERS</a> window, click <a href="#">INVITE A PARTNER</a>.</li> <li>• In the email field enter: <a href="mailto:3D@Perio-Protect.com">3D@Perio-Protect.com</a></li> <li>• Click <a href="#">SEND</a>.</li> </ul> <p>Once we accept, you may start sending cases</p>	<p><b>Other Scanning Software</b> Upload STL files directly through our website. Go to: <a href="http://www.PerioProtect.com/3D">www.PerioProtect.com/3D</a></p> <ul style="list-style-type: none"> <li>• Click <a href="#">UPLOAD</a>.</li> <li>• Click <a href="#">CHOOSE FILES</a>.</li> <li>• Select file, enter patient name or chart number, doctor or office name and office email.</li> <li>• Click <a href="#">UPLOAD</a>.</li> </ul> <p>This method is HIPAA compliant and allows you to send STL files, Rx and perio charting at the same time.</p>
<p><b>iTero</b> Call iTero at <a href="tel:800-577-8767">800-577-8767</a> (ext 1 or 2) to add Perio Protect Lab as a preferred lab. Or find us in iTero by our ID: <a href="#">38291</a>.</p>	<p><b>Send Rx &amp; Periodontal Charting</b> You have 3 OPTIONS:</p> <ul style="list-style-type: none"> <li>• <a href="#">Fax</a> both to 888-511-2308.</li> <li>• <a href="#">Upload</a> both files directly to our website. Go to: <a href="http://www.PerioProtect.com/3D">www.PerioProtect.com/3D</a> Click <a href="#">UPLOAD</a>, then <a href="#">CHOOSE FILES</a>. Select files, enter doctor's full name and office email. Then click <a href="#">UPLOAD</a>.</li> <li>• <a href="#">Email script or probing chart to lab:</a> <a href="mailto:lab@perio-protect.com">lab@perio-protect.com</a> OR <a href="mailto:PerioProtect3D@gmail.com">PerioProtect3D@gmail.com</a></li> </ul>
<p><b>Medit</b> Go the Medit website to add Perio Protect Lab as a partner:</p> <ul style="list-style-type: none"> <li>• Click on <a href="#">PARTNERS</a> in the navigation bar on the left.</li> <li>• Under <a href="#">ADD NEW PARTNERS</a>, in the <a href="#">SEARCH BY CLINIC NAME, EMAIL ADDRESS</a> window, enter our email address: <a href="mailto:3D@Perio-Protect.com">3D@Perio-Protect.com</a></li> </ul> <p>If Medit does not find us, click on the blue <a href="#">INVITE PARTNER</a> button on the right and enter our email address (above).</p>	<p><b>Tips and File Preferences</b></p> <ul style="list-style-type: none"> <li>• 3D scans must be in STL format.</li> <li>• File per arch. We prefer arches oriented in occlusion.</li> </ul>
<p><b>Planmeca</b></p> <ul style="list-style-type: none"> <li>• Export 3D scan from Planmeca and save as 3Shape (preferred) or as an STL file.</li> </ul> <p>Go to: <a href="http://www.PerioProtect.com/3D">www.PerioProtect.com/3D</a>.</p> <ul style="list-style-type: none"> <li>• Click <a href="#">UPLOAD</a>, then <a href="#">CHOOSE FILES</a>.</li> <li>• Select file, enter doctor's full name and office email.</li> <li>• Click <a href="#">+ADD MORE FILES</a> to upload Rx and perio charting.</li> <li>• Click <a href="#">UPLOAD</a>.</li> </ul>	

Perio Protect Laboratory  
9701 Green Park Industrial Drive  
St. Louis, MO 63123

**Phone:** 314-474-0467  
**Fax:** 888-511-2308  
**Email:** [lab@perio-protect.com](mailto:lab@perio-protect.com)  
**Web:** [perioprotect.com/3D](http://perioprotect.com/3D)