

Restore Structure Relieve Hypersensitivity

ReminGel™

Retains
Natural
Mineral
Balance

ReminGel™ is a patent-pending, convenient, ready-to-use gel, consisting of a blend of hydroxyapatite, hydroxyapatite precursors, and free calcium and phosphate that help restore the tooth's hydroxyapatite structure through remineralization. This results in tubule occlusion and blocks fluid flow in dentinal tubules, relieving dentinal sensitivity and promoting ongoing remineralization of the affected area.

Clinical Case: ReminGel treatment with PerioProtect Trays® daily for 15 minutes. No prior treatment, routine cleaning only.



Day 0



3 month Recall
Courtesy of Dr. Y. Peterson

Key Benefits:

- Relieve aching procedures, cold, heat, acids or sweets.
- Helps remineralize enamel, reduces white spot lesions, and delivers minerals where they are needed most.^{1,2,3}
- Biocompatible, fluoride-free, and non-irritating - suitable for both in-office application and at-home use.⁴



Patient Centered - Flexible Delivery

ReminGel™ is a scientifically-formulated aqueous gel. The mechanism of action includes both physical tubule occlusion and potassium-mediated nerve desensitization.


ReminGel has a hydroxyapatite-rich formulation that is enhanced with buffering agents and flavoring to improve patient compliance and comfort. With an application time of 3-15 minutes with PerioProtect Trays®, ReminGel is easily incorporated into existing treatment protocols and into patients daily oral hygiene routines. It is a versatile option for managing severe tooth sensitivity, across a range of treatment scenarios, including post-whitening care, while actively supporting tooth remineralization.

For prescription and over-the-counter use supporting both clinical efficacy and patient accessibility.



ReminGel, 3 oz. RGEL-3000

Laser Scanning Microscope of Dentin Surface

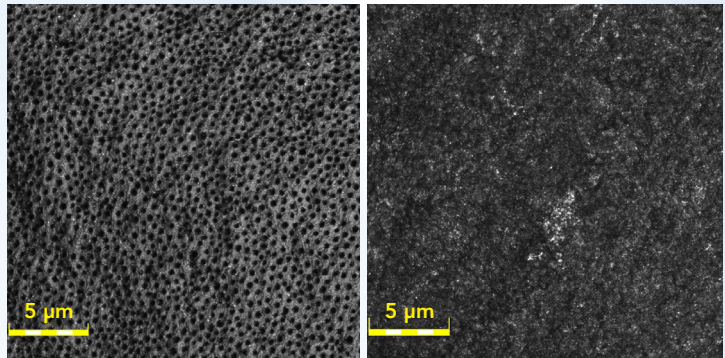

Untreated

ReminGel (7 applications for 30 minutes, complete tubule occlusion)

PerioProtect Trays®:

- Customized, non-invasive and comfortable
- Designed to hold medication where it is most effective
- The ideal delivery system for ReminGel



References:

1. WSL case studies on file.
2. Dentin occlusion data on file.
3. O'Hagan-Wong K, Enax J, Meyer F, Ganss B. The use of hydroxyapatite toothpaste to prevent dental caries. *Odontology*. 2022 Apr;110(2):223-230. doi: 10.1007/s10266-021-00675-4. Epub 2021 Nov 22. PMID: 34807345; PMCID: PMC8930857.
4. ISO 10993 biocompatibility testing on file.

VistaApex Solutions

2200 South Street
Racine, WI 53404

Phone: 877.434.GUMS (4867)
Email: support@perio-protect.com
Website: <https://www.perioprotect.com>

Scan here for more
informaton:

